TRS 130

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Table of contents

Preface.....	9
Declaration of conformity.....	9
Service life.....	9
Disposal.....	9
Identification plate.....	9
Reuse.....	9
Warranty.....	9
Parts of the product.....	10
Indication.....	11
Scope of delivery.....	11
Safety instructions.....	11
Assembly.....	11
Locking and unlocking the armrests.....	11
Double-function parking brake.....	12
Mounting the footrests.....	12
Locking and unlocking the footrests.....	12
Toilet bucket.....	12
Height adjustment of the footrests.....	12
Cleaning Maintenance.....	13
Technical data.....	13

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co.KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.

This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year/month)

Reuse

This product can be reused. Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available at www.drivedevilbiss.de or from your specialist dealer.

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes and handles

Parts of the product

- | | | |
|-------------------------|---|---------------------------------------|
| 1. Push handle | 5. Armrest locking mechanism | 9. Heel strap |
| 2. Back rest | 6. Double-function parking brake | 10. Footrest locking mechanism |
| 3. Locking screw | 7. Toilet bucket with cover | 11. Cover plate |
| 4. Seat | 8. Footrests | 12. Armrest |

Indication


INDICATION:

Significant to full impairment of mobility/walking and transfer in the case of structural and/or functional damage to the lower extremities (including amputation, after-effects of injuries, musculoskeletal/neuromusculoskeletal movement disorders)

- For use of the integrated toilet unit or to position over a toilet.

Scope of delivery

1 x A	Wheeled commode
1 x B	Bucket with cover
2 x C	Footrests
1 x D	Heel strap
1 x E	Seat cover
1 x F	Backrest with armrests
1 x	Instructions for use

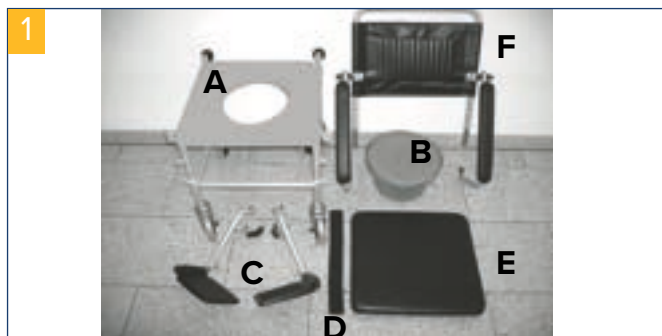Safety instructions


- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Clothing or body parts can be pinched when sitting down, standing up, inserting the toilet bucket, as well as when raising the armrests and locking them into place.
- If you need to move over thresholds, do so slowly and carefully, otherwise the wheels can become damaged!
- **IMPORTANT:** If you need to move in reverse over thresholds with a person sitting in the wheeled commode, there is a risk of the armrest lock coming loose.
- Avoid wet conditions – risk of slipping!
- When parking a wheeled commode, always use the parking brakes.
- During transfer into/out of the wheeled commode, the footrests must be swiveled out of the way and the locking brakes must be applied.

- Never use the footrests as assistance when getting up – risk of tipping!
- The wheeled commode is significantly less stable than a regular wheelchair, therefore avoid leaning to the side – risk of tipping!
- Prevent unauthorized use, for example, by children.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.

Assembly

The wheeled commode can be easily assembled.

Place the preassembled back unit on the base frame and secure it by turning the star screws clockwise (**photo 2**).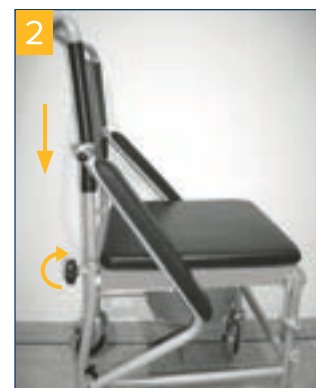


Locking and unlocking the armrests

Pull both armrests upwards until they audibly engage (**photo 3**).

They will automatically click into place. Ensure that they successfully click into place on both sides; in this case, the locking pin (1) must be visible (**photo 4**).

To unlock the armrests, pull the locking knob back (2) and swing the armrest downwards (**photo 4**).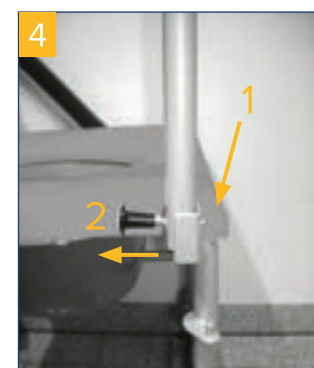

IMPORTANT: It is possible for the mounting of the backrest to become slightly bent due to transport; this can be easily bent back into place until the locking knob clicks correctly into place in the front.

Double-function parking brake


To apply the brake, press the lever downwards (**photo 5**). At the same time, the directional position of the wheel is blocked.

To release the brake, press the head of the lever downwards (**photo 6**).

Mounting the footrests

Hang the provided footrests with the two holes on the respective retaining pins as shown (photo 7+8).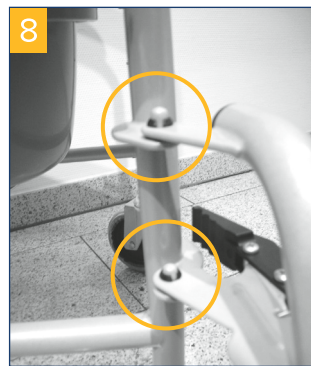


Locking and unlocking the footrests

To lock the footrests, swivel them to the center until they audibly click into place. To unlock them, press the lever inwards as shown and move the footrests to the side (**photo 9**).

Toilet bucket

Always insert and remove the toilet bucket from the rear (also with the cover on). The toilet bucket snaps into place and the toilet seat can be easily lifted off for cleaning (**photo 10**).

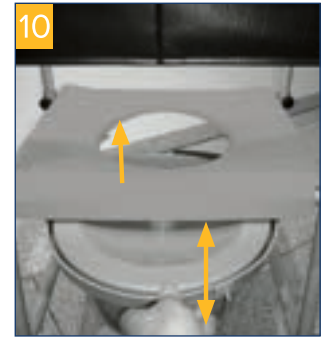Height adjustment of the footrests

To adjust the height of the footrests, first loosen the clamping screw (**photo 11**) with the wrench provided. After shifting, the footrests can be adjusted (**photo 12**) for lower leg lengths of 420 – 500 mm.

Tighten the clamping screw. The heel strap can optionally be attached using the Velcro fastener (**photo 13**).

IMPORTANT:

Never use footrests to assist with standing up!

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.
- Check the condition of the tires regularly. They are "fail-safe", meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage, however.

Technical data

Product	TRS 130
Article number	530100000
HMV (medical aids register) no.	18.46.02.0058
Overall length	95 cm
Overall length without footrests	63 cm
Overall width	56 cm
Overall height	97 cm
Height of backrest	38 cm
Back angle	20.5 cm
Seat width	44 cm
Seat depth	42.5 cm
Seat height	52 cm
Height between floor and seat	43 cm
Weight	14.4 kg
Max. load	130 kg
Frame material	Welded steel tubing, powder-coated
Seat / armrest cover material	PP (polypropylene) / PP (polyurethane)

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Drive Medical GmbH & Co. KG